

**snogg®**

ANIMAL CARE



# —Animal— **WOUND CARE**

FOR TREATMENT AND SUPPORT





# Animal Wound Care since 1980

**Snögg Animal Care** has more than 40 years of experience of animal wound care. Developed in co-operation with veterinarians and animal wound care experts, the products are intended for animal use – with a focus on animal well-being. The product range comprises different bandage types that are all easily adapted to different needs and body parts, offering the user creative freedom.



**Where can our *Adhesive Foam Bandages* be used?** Find out the answer and more on our website. Scan the QR code to come to FAQ!

**New**



**Snögg** is expanding its range with three dressings for different types of wounds, including small wounds, heavily exuding wounds and wounds at risk of infection. The three primary dressings are suitable for fixing with **Adhesive Foam Bandages**.





- ✓ Easy to apply and remove – without unnecessary stress for the animal
- ✓ Can be used to bandage difficult areas of the animal's body
- ✓ Multi-use – combine and customize for all animal types and stages of the wound healing process
- ✓ Can easily be combined with other wound care procedures
- ✓ For treatment and support
- ✓ Packaging design with clear instructions





# Primary layers

**Snogg** is expanding its range with three new primary dressings to meet more needs in wound care. The new dressings are produced to handle different types of wounds and conditions: small wounds, heavily exuding wounds and wounds at risk of infection. They work very well by combining with other wound care products and are suitable for fixing with **Adhesive Foam Bandages**.

## BactiDefend DRESSING

New

### REDUCES BIOBURDEN UP TO 99%

A sterile hydrogel dressing using peptides mimicking the immune system. The dressing binds and physically disrupts bacteria, without any leaching of substances. The dressing also promotes wound healing by providing a moist environment. Safe and biocompatible. For professional use only. Use on dry or low exuding wounds.

- Uses peptides, mimicking the immune system
- Binds and physically disrupts bacteria
- Promotes wound healing



**REF 510003786**  
5 pieces (4.5 x 15 cm)



# Foam COMPRESS



## PROMOTES WOUND HEALING

A sterile and absorbent foam compress that keeps wound moist and promotes wound healing. A soft and flexible dressing that follows the body's movement. It does not stick to the wound and can be cut to desired size.

- Sterile and absorbent
- Providing moist wound healing
- Soft and flexible



REF 510003122  
10 pieces (10 x 10 cm)

REF 510003123  
10 pieces (5 x 5 cm)

# Superabsorbent COMPRESS



## FOR HEAVILY EXUDING WOUNDS

A sterile compress with high absorbent capacity for heavily exuding wounds. The internal superabsorbent core can absorb large amount of exudate. The exudate remains locked within the dressing pad to help prevent exudate from leaking through the dressing.

- Sterile
- High absorbent capacity
- Helps to prevent leaking of exudate



REF 510003124  
10 pieces (10 x 10 cm)

REF 510003125  
10 pieces (10 x 20 cm)

# Adhesive FOAM BANDAGE

*Adhesive Foam Bandage* is an elastic foam dressing with great adhesion to both fur and skin. The dressing has adhesion on one side and is ideal for padding and fixation or for monitoring devices, i.e. ECG-pads. You can easily create a wound bandage by combining *Adhesive Foam Bandage* with a compress, or *Thick Foam Bandage*.

**LATEX  
FREE**

- Soft and flexible
- Adhesion on one side
- Can be applied anywhere on the animal



*When you need great  
adhesion to fur*



## Adhesive Foam Bandage is available in 3 sizes to suit different animals and wounds:



**REF 510002410**  
(Formerly Animal Polster)  
9 x 200 cm



**REF 510002412**  
(Formerly Animal Polster)  
14 x 200 cm




**REF 510002413**  
(Formerly Animal Polster)  
22 x 200 cm

### BANDAGING WOUNDS:



Ensure the wound is clean and the fur is dry.

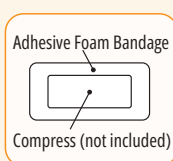

Use a compress (not included) to protect the wound.



Fix the compress with the Adhesive Foam Bandage.



Scan the QR code for full instructions.

## Adhesive Foam Bandage — REMOVAL WIPES

The **Removal Wipes** offer an easier and gentler way to remove glue residuals from the animal's fur reducing discomfort or irritation. Non-alcoholic formula that does not dry out the skin. The wipes are a complement to **Adhesive Foam Bandages**.

### GENTLE REMOVAL OF ADHESIVE FOAM BANDAGES:

- Ease removal
- Easy and effective to use
- Non-alcoholic formula

**LATEX FREE**



**REF 510002378**  
30 wipes (12 x 18 cm)





*Flexibility and freedom  
to move with Thick  
Foam Bandage*



# Thick FOAM BANDAGE

**Thick Foam Bandage** is a unique self-adherent bandage; it sticks to itself but does not stick to the wound or the fur nor the skin. It is flexible, provides great mobility and is easy to shape. The absorbing and soft foam bandage is ideal to be used on wounds for fixation or padding.


**LATEX  
FREE**

- Only sticks to itself
- Can be placed directly on the wound
- Absorbing, soft and flexible




**REF 510002411**  
(Formerly Animal Vet)  
9 x 200 cm


## PAW BANDAGING:

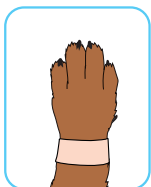


Wrap one length of Thick or Thin Foam Bandage around the leg.

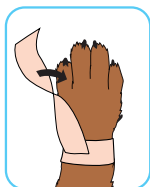

Tear off a suitable length to fit between the digits. Fix it to the wrap, turn it 180 degrees.

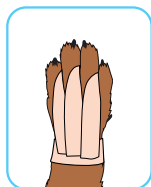

Slip the narrow part between the digits and fix the wrap on the other side. Repeat for each digit.

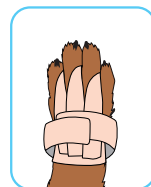

Complete by wrapping a second bandage around the leg to secure the bandage.


## LEG BANDAGING:



Ensure the wound is clean and fur/leg is dry.



Begin by wrapping one round of Adhesive Foam Bandage (not included) below the wounded area for fixation.

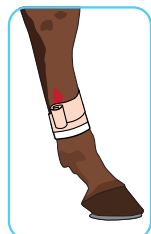

Then wrap the Thick Foam Bandage around the wound in a spiral pattern. Overlap the first bandage by 2–3 cm.



Replace the second bandage when required. The first bandage is removed when healing is completed.



Scan the QR code for full instructions.



# Thin FOAM BANDAGE

*Thin Foam Bandage* is easy to shape and provides great flexibility. The cross-elastic bandage sticks to itself and does not stick to the skin or the fur nor the wound. It is soft, absorbent and can be placed directly on the wound.


LATEX  
FREE

- Only sticks to itself
- Can be placed directly on the wound
- Many usage areas, for example cow teats




REF 510002379  
(Formerly Animal Soft)  
6 x 450 cm


## PAW BANDAGING:

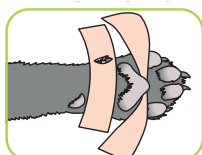


Place a length of bandage between the digits and the pad. Make a slot in the bandage and place it around the dew claw.

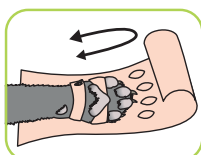

Tear off a strip of bandage to length. Make slots to go over the digits.

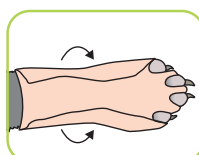

Slip the digits through the slots and fold the bandage as shown.


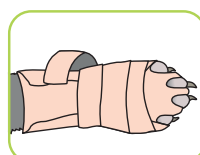


If necessary, wrap an additional length of bandage around the paw to secure the bandage.



Scan the QR code for full instructions.

## TEAT BANDAGING:

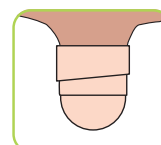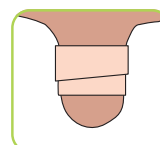

Use Thin Foam Bandage in the event of injury of udders or fixation of teat plugs.







*Elastic and  
self-adherent  
Thin Foam Bandage*

## Flex WRAP BANDAGE

*Flex Wrap Bandage* is a flexible and elastic outer wrap bandage. It does not stick to skin, fur nor wounds and is ideal for fixation of compresses and self-adherent Foam Bandages. Protects from dirt and moisture and fits on many parts of the animal body. Available in 2 sizes, packed in a practical cardboard display.

- Easy and quick to apply and remove
- Sticks to itself
- Protects from dirt and moisture



REF 510002414  
5 x 450 cm

REF 510002393  
10 x 450 cm



# Product overview

Ease  
removal of  
Adhesive Foam  
Bandage with  
**REMOVAL  
WIPES**



	ADHESIVE FOAM BANDAGE	THICK FOAM BANDAGE	THIN FOAM BANDAGE	FLEX WRAP BANDAGE
Sticks to itself – not to fur, skin or wounds	✗	✓	✓	✓
Excellent adhesion to fur & skin	✓	✗	✗	✗
Cross-elastic	✓	✓	✓	✗
Length-elastic	✗	✓	✓	✓
Absorbent	✗	✓	✓	✗
Stops bleeding	✗	✓	✓	✗
Adherent even when wet	✗	✓	✓	✗
Suitable for primary layers	✗	✓	✓	✗
Suitable for secondary layers	✓	✓	✓	✓
Suitable for tertiary layers	✓	✗	✗	✓
Fixation	✓	✓	✓	✓
Thickness	5 mm	5 mm	1,9 mm	

New



	BACTIDEFEND DRESSING	FOAM COMPRESS	SUPERABSORBENT COMPRESS
Low exuding wounds	✓	✓	✗
Medium exuding wounds	✗	✓	✓
Heavily exuding wounds	✗	✗	✓
Wounds at risk of infection	✓	✗	✗
Surgical wounds	✓	✓	✓
Binds and physically disrupts bacteria	✓	✗	✗
Moist wound healing	✓	✓	✗
Sterile	✓	✓	✓
Can be cut to desired size	✓	✓	✗
Suitable for fixing with Adhesive Foam Bandages	✓	✓	✓

[www.snogg.com](http://www.snogg.com)

Orkla Wound Care AB (Svetsarvägen 15), Box 1336, SE-171 26 Solna, Sweden  
Tel +46 (0)8 590 96325, [snogganimal@orkla.com](mailto:snogganimal@orkla.com), [www.snogg.com](http://www.snogg.com)