

snogg®

ANIMAL CARE



— Animal — WOUND CARE

A WOUND CARE CONCEPT FOR ANIMALS



- ✓ Easy to apply and remove
– without unnecessary stress for the animal
- ✓ Can be used to bandage difficult areas of the animal's body
- ✓ Multi-use – combine and customize for all animal types and stages of the wound healing process
- ✓ For treatment and support
- ✓ Packaging design with clear instructions



Animal Wound Care since 1980

Snögg Animal Care has more than 40 years of experience of animal wound care. Developed in co-operation with veterinarians and animal wound care experts, the products are intended for animal use – with a focus on animal well-being. The product range comprises different bandage types that are all easily adapted to different needs and body parts, offering the user creative freedom.

Adhesive FOAM BANDAGE

**LATEX
FREE**

WHEN YOU NEED GREAT ADHESION TO FUR

- Adhesion on one side
- Ideal for padding and fixing
- Soft and flexible




REF 510002410
(Formerly Animal Polster)
9 x 200 cm


REF 510002412
(Formerly Animal Polster)
14 x 200 cm

REF 510002413
(Formerly Animal Polster)
22 x 200 cm


Lägga sårförband:

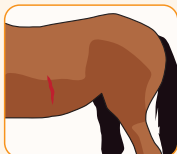

Ensure the wound is clean and the fur is dry.

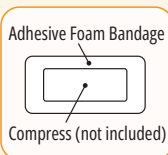

Use a compress (not included) to protect the wound.

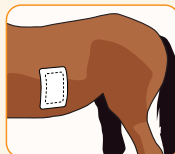

Fix the compress with the Adhesive Foam Bandage.



Scan the QR code for full instructions.

Adhesive Foam Bandage — REMOVAL WIPES

GENTLE REMOVAL OF ADHESIVE FOAM BANDAGES:

- Ease removal
- Easy and effective to use
- Non-alcoholic formula



REF 510002378
30 wipes (12 x 18 cm)

Thick & Thin FOAM BANDAGE

LATEX
FREE

FOR FLEXIBILITY AND FREEDOM OF MOVEMENT

- Only sticks to itself
- Can be placed directly on the wound
- Absorbing, soft and flexible



REF 510002411
(Formerly Animal Vet)
9 x 200 cm



REF 510002379
(Formerly Animal Soft)
6 x 450 cm

Paw bandaging:




Wrap one length of Thick or Thin Foam Bandage around the leg.



Tear off a suitable length to fit between the digits. Fix it to the wrap, turn it 180 degrees.



Slip the narrow part between the digits and fix the wrap on the other side. Repeat for each digit.

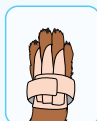

Complete by wrapping a second bandage around the leg to secure the bandage.



Scan the QR code for full instructions.

Flex WRAP BANDAGE

AN OUTER DRESSING FOR FIXING

- Easy and quick to apply and remove
- Sticks to itself
- Protects from dirt and moisture



REF 510002414
5 x 450 cm

REF 510002393
10 x 450 cm





Ease
removal of
Adhesive Foam
Bandage with
**REMOVAL
WIPES**



OVERVIEW	ADHESIVE FOAM BANDAGE	THICK FOAM BANDAGE	THIN FOAM BANDAGE	FLEX WRAP BANDAGE
Sticks to itself – not to fur, skin or wounds	✗	✓	✓	✓
Excellent adhesion to fur & skin	✓	✗	✗	✗
Cross-elastic	✓	✓	✓	✗
Length-elastic	✗	✓	✓	✓
Absorbent	✗	✓	✓	✗
Stops bleeding	✗	✓	✓	✗
Adherent even when wet	✗	✓	✓	✗
Suitable for primary layers	✗	✓	✓	✗
Suitable for secondary layers	✓	✓	✓	✓
Suitable for tertiary layers	✓	✗	✗	✓
Fixation	✓	✓	✓	✓
Thickness	5 mm	5 mm	1,9 mm	

Primary layers

Snøgg is expanding its range with three dressings for different types of wounds, including small wounds, heavily exuding wounds and wounds at risk of infection. The three primary dressings are suitable for fixing with **Adhesive Foam Bandage**.

Foam COMPRESS



PROMOTES WOUND HEALING

- Sterile and absorbent
- Providing moist wound healing
- Soft and flexible

New



REF 510003122
10 pieces (10 x 10 cm)

REF 510003123
10 pieces (5 x 5 cm)

Superabsorbent COMPRESS



FOR HEAVILY EXUDING WOUNDS

- Sterile
- High absorbent capacity
- Helps to prevent leaking of exudate

New



REF 510003124
10 pieces (10 x 10 cm)

REF 510003125
10 pieces (10 x 20 cm)

BactiDefend DRESSING

REDUCES BIOBURDEN UP TO 99%

New

- Uses peptides, mimicking the immune system
- Binds and physically disrupts bacteria
- Promotes wound healing



REF 510003786
5 pieces (4.5 x 15 cm)



OVERVIEW

	BACTIDEFEND DRESSING	FOAM COMPRESS	SUPERABSORBENT COMPRESS
Low exuding wounds	✓	✓	✗
Medium exuding wounds	✗	✓	✓
Heavily exuding wounds	✗	✗	✓
Wounds at risk of infection	✓	✗	✗
Surgical wounds	✓	✓	✓
Binds and physically disrupts bacteria	✓	✗	✗
Moist wound healing	✓	✓	✗
Sterile	✓	✓	✓
Can be cut to desired size	✓	✓	✗
Suitable for fixing with Adhesive Foam Bandages	✓	✓	✓



snogg®

ANIMAL CARE

www.snogg.com

Orkla Wound Care AB (Svetsarvägen 15),
Box 1336, SE-171 26 Solna, Sweden
Tel +46 (0)8 590 96325,
snogganimal@orkla.com