

snogg®

ANIMAL CARE



— Animal — WOUND CARE

FOR TREATMENT AND SUPPORT





Animal Wound Care since 1980

Snögg Animal Care has more than 40 years of experience of animal wound care. Developed in co-operation with veterinarians and animal wound care experts, the products are intended for animal use – with a focus on animal well-being. The product range comprises various types of bandages and dressings that are all easily adapted to different needs, areas of the body, and types of wounds, offering the user creative freedom.



Where can our Adhesive Foam Bandages be used? Find out the answer and more on our website. Scan the QR code to come to FAQ!





- ✓ Easy to apply and remove - without unnecessary stress for the animal
- ✓ Can be used to bandage difficult areas of the animal's body
- ✓ Multi-use - combine and customize for all animal types and stages of the wound healing process
- ✓ Can easily be combined with other wound care procedures
- ✓ For treatment and support
- ✓ Packaging design with clear instructions



Primary layers

Snøgg Animal Care offers two types of dressings for different kinds of wounds, including small wounds and heavily exuding wounds. They work very well by combining with other wound care products and are suitable for fixing with *Adhesive Foam Bandage*.

Foam COMPRESS



PROMOTES WOUND HEALING

A sterile and absorbent foam compress that keeps wound moist and promotes wound healing. A soft and flexible dressing that follows the body's movement. It does not stick to the wound and can be cut to desired size.

- Sterile and absorbent
- Providing moist wound healing
- Soft and flexible



REF 510003122
10 pieces (10 x 10 cm)

REF 510003123
10 pieces (5 x 5 cm)

Superabsorbent COMPRESS



FOR HEAVILY EXUDING WOUNDS

A sterile compress with high absorbent capacity for heavily exuding wounds. The internal superabsorbent core can absorb large amount of exudate. The exudate remains locked within the dressing pad to help prevent exudate from leaking through the dressing.

- Sterile
- High absorbent capacity
- Helps to prevent leaking of exudate



REF 510003124
10 pieces (10 x 10 cm)

REF 510003125
10 pieces (10 x 20 cm)

Adhesive FOAM BANDAGE

Adhesive Foam Bandage is an elastic foam dressing with great adhesion to both fur and skin. The dressing has adhesion on one side and is ideal for padding and fixation or for monitoring devices, i.e. ECG-pads. You can easily create a wound bandage by combining *Adhesive Foam Bandage* with a compress, or *Thick Foam Bandage*.

LATEX
FREE

- Soft and flexible
- Adhesion on one side
- Can be applied anywhere on the animal



*When you need great
adhesion to fur*

Adhesive Foam Bandage is available in 3 sizes to suit different animals and wounds:



REF 510002410
(Formerly Animal Polster)
9 x 200 cm




REF 510002412
(Formerly Animal Polster)
14 x 200 cm




REF 510002413
(Formerly Animal Polster)
22 x 200 cm


BANDAGING WOUNDS:

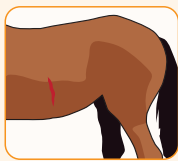

Ensure the wound is clean and the fur is dry.

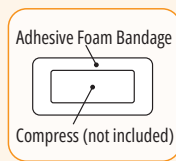

Use a compress (not included) to protect the wound.

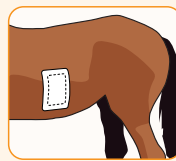

Fix the compress with the Adhesive Foam Bandage.



Scan the QR code for full instructions.

Adhesive Foam Bandage REMOVAL WIPES

The *Removal Wipes* offer an easier and gentler way to remove glue residuals from the animal's fur reducing discomfort or irritation. Non-alcoholic formula that does not dry out the skin. The wipes are a complement to *Adhesive Foam Bandages*.

GENTLE REMOVAL OF ADHESIVE FOAM BANDAGES:

- Ease removal
- Easy and effective to use
- Non-alcoholic formula

LATEX FREE



REF 510002378
30 wipes (12 x 18 cm)



*Flexibility and freedom
to move with Thick
Foam Bandage*

Thick FOAM BANDAGE

Thick Foam Bandage is a unique self-adherent bandage; it sticks to itself but does not stick to the wound or the fur nor the skin. It is flexible, provides great mobility and is easy to shape. The absorbing and soft foam bandage is ideal to be used on wounds for fixation or padding.


LATEX
FREE

- Only sticks to itself
- Can be placed directly on the wound
- Absorbing, soft and flexible




REF 510002411
(Formerly Animal Vet)
9 x 200 cm


PAW BANDAGING:

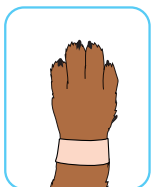


Wrap one length of Thick or Thin Foam Bandage around the leg.

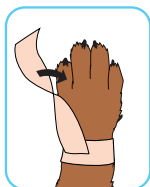

Tear off a suitable length to fit between the digits. Fix it to the wrap, turn it 180 degrees.

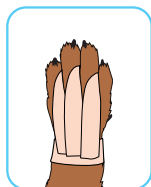

Slip the narrow part between the digits and fix the wrap on the other side. Repeat for each digit.

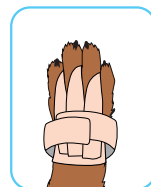

Complete by wrapping a second bandage around the leg to secure the bandage.


LEG BANDAGING:



Ensure the wound is clean and fur/leg is dry.



Begin by wrapping one round of Adhesive Foam Bandage (not included) below the wounded area for fixation.

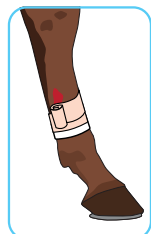

Then wrap the Thick Foam Bandage around the wound in a spiral pattern. Overlap the first bandage by 2–3 cm.



Replace the second bandage when required. The first bandage is removed when healing is completed.



Scan the QR code for full instructions.

Thin FOAM BANDAGE

Thin Foam Bandage is easy to shape and provides great flexibility. The cross-elastic bandage sticks to itself and does not stick to the skin or the fur nor the wound. It is soft, absorbent and can be placed directly on the wound.


LATEX
FREE

- Only sticks to itself
- Can be placed directly on the wound
- Many usage areas, for example cow teats




REF 510002379
(Formerly Animal Soft)
6 x 450 cm


PAW BANDAGING:

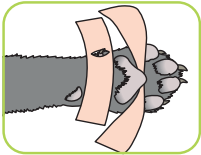


Place a length of bandage between the digits and the pad. Make a slot in the bandage and place it around the dew claw.

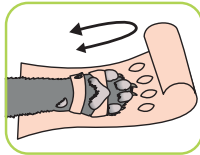

Tear off a strip of bandage to length. Make slots to go over the digits.

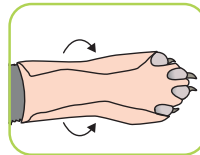

Slip the digits through the slots and fold the bandage as shown.


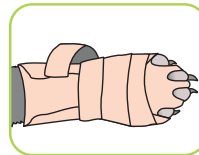


If necessary, wrap an additional length of bandage around the paw to secure the bandage.



Scan the QR code for full instructions.

TEAT BANDAGING:

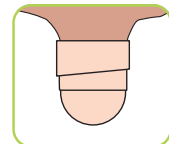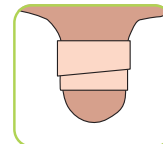

Use Thin Foam Bandage in the event of injury of udders or fixation of teat plugs.





*Elastic and
self-adherent
Thin Foam Bandage*

Flex WRAP BANDAGE

Flex Wrap Bandage is a flexible and elastic outer wrap bandage. It does not stick to skin, fur nor wounds and is ideal for fixation of compresses and self-adherent Foam Bandages. Protects from dirt and moisture and fits on many parts of the animal body. Available in 2 sizes, packed in a practical cardboard display.

- Easy and quick to apply and remove
- Sticks to itself
- Protects from dirt and moisture



REF 510002414
5 x 450 cm

REF 510002393
10 x 450 cm

Product overview

Ease removal of Adhesive Foam Bandage with **REMOVAL WIPES**



	ADHESIVE FOAM BANDAGE	THICK FOAM BANDAGE	THIN FOAM BANDAGE	FLEX WRAP BANDAGE
Sticks to itself – not to fur, skin or wounds	✗	✓	✓	✓
Excellent adhesion to fur & skin	✓	✗	✗	✗
Cross-elastic	✓	✓	✓	✗
Length-elastic	✗	✓	✓	✓
Absorbent	✗	✓	✓	✗
Stops bleeding	✗	✓	✓	✗
Adherent even when wet	✗	✓	✓	✗
Suitable for primary layers	✗	✓	✓	✗
Suitable for secondary layers	✓	✓	✓	✓
Suitable for tertiary layers	✓	✗	✗	✓
Fixation	✓	✓	✓	✓
Thickness	5 mm	5 mm	1,9 mm	



	FOAM COMPRESS	SUPERABSORBENT COMPRESS
Low exuding wounds	✓	✗
Medium exuding wounds	✓	✓
Heavily exuding wounds	✗	✓
Surgical wounds	✓	✓
Moist wound healing	✓	✗
Sterile	✓	✓
Can be cut to desired size	✓	✗
Suitable for fixing with Adhesive Foam Bandages	✓	✓

www.snogg.com

Orkla Wound Care AB (Svetsarvägen 15), Box 1336, SE-171 26 Solna, Sweden
Tel +46 (0)8 590 96325, snogganimal@orkla.com, www.snogg.com