

**snogg®**

ANIMAL CARE



# Animal WOUND CARE

A WOUND CARE CONCEPT FOR ANIMALS



- ✓ Easy to apply and remove  
– without unnecessary stress for the animal
- ✓ Can be used to bandage difficult areas of the animal's body
- ✓ Multi-use – combine and customize for all animal types and stages of the wound healing process
- ✓ For treatment and support
- ✓ Packaging design with clear instructions



## Animal Wound Care since 1980

*Snøgg Animal Care* has more than 40 years of experience of animal wound care. Developed in co-operation with veterinarians and animal wound care experts, the products are intended for animal use – with a focus on animal well-being. The product range comprises different bandage types that are all easily adapted to different needs and body parts, offering the user creative freedom.

# Adhesive FOAM BANDAGE

LATEX  
FREE

## WHEN YOU NEED GREAT ADHESION TO FUR

- Adhesion on one side
- Ideal for padding and fixing
- Soft and flexible




**REF 510002410**  
(Formerly Animal Polster)  
9 x 200 cm


**REF 510002412**  
(Formerly Animal Polster)  
14 x 200 cm

**REF 510002413**  
(Formerly Animal Polster)  
22 x 200 cm


## Bandaging wounds:

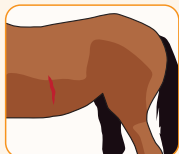

Ensure the wound is clean and the fur is dry.

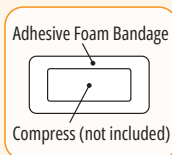

Use a compress (not included) to protect the wound.

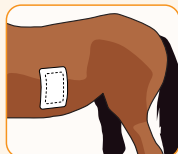

Fix the compress with the Adhesive Foam Bandage.



Scan the QR code for full instructions.

# Adhesive Foam Bandage REMOVAL WIPES

## GENTLE REMOVAL OF ADHESIVE FOAM BANDAGES:

- Ease removal
- Easy and effective to use
- Non-alcoholic formula



**REF 510002378**  
30 wipes (12 x 18 cm)

# Thick & Thin FOAM BANDAGE

LATEX  
FREE

## FOR FLEXIBILITY AND FREEDOM OF MOVEMENT

- Only sticks to itself
- Can be placed directly on the wound
- Absorbing, soft and flexible



**REF 510002411**  
*(Formerly Animal Vet)*  
9 x 200 cm



**REF 510002379**  
*(Formerly Animal Soft)*  
6 x 450 cm

### Paw bandaging:




Wrap one length of Thick or Thin Foam Bandage around the leg.



Tear off a suitable length to fit between the digits. Fix it to the wrap, turn it 180 degrees.



Slip the narrow part between the digits and fix the wrap on the other side. Repeat for each digit.

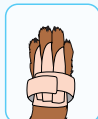

Complete by wrapping a second bandage around the leg to secure the bandage.



Scan the QR code for full instructions.

# Flex WRAP BANDAGE

## AN OUTER DRESSING FOR FIXING

- Easy and quick to apply and remove
- Sticks to itself
- Protects from dirt and moisture



**REF 510002414**  
5 x 450 cm

**REF 510002393**  
10 x 450 cm



Ease  
removal of  
Adhesive Foam  
Bandage with  
**REMOVAL  
WIPES**



OVERVIEW	ADHESIVE FOAM BANDAGE	THICK FOAM BANDAGE	THIN FOAM BANDAGE	FLEX WRAP BANDAGE
Sticks to itself – not to fur, skin or wounds	✗	✓	✓	✓
Excellent adhesion to fur & skin	✓	✗	✗	✗
Cross-elastic	✓	✓	✓	✗
Length-elastic	✗	✓	✓	✓
Absorbent	✗	✓	✓	✗
Stops bleeding	✗	✓	✓	✗
Adherent even when wet	✗	✓	✓	✗
Suitable for primary layers	✗	✓	✓	✗
Suitable for secondary layers	✓	✓	✓	✓
Suitable for tertiary layers	✓	✗	✗	✓
Fixation	✓	✓	✓	✓
Thickness	5 mm	5 mm	1,9 mm	

# Primary layers

*Snøgg Animal Care* offers two types of dressings for different kinds of wounds, including small wounds and heavily exuding wounds.

The primary dressings are suitable for fixing with *Adhesive Foam Bandage*.

## Foam COMPRESS



### PROMOTES WOUND HEALING

- Sterile and absorbent
- Providing moist wound healing
- Soft and flexible



REF 510003122  
10 pieces (10 x 10 cm)

REF 510003123  
10 pieces (5 x 5 cm)

# — Superabsorbent — COMPRESS



## FOR HEAVILY EXUDING WOUNDS

- Sterile
- High absorbent capacity
- Helps to prevent leaking of exudate



**REF 510003124**  
10 pieces (10 x 10 cm)

**REF 510003125**  
10 pieces (10 x 20 cm)



## OVERVIEW

	FOAM COMPRESS	SUPERABSORBENT COMPRESS
Low exuding wounds	✓	✗
Medium exuding wounds	✓	✓
Heavily exuding wounds	✗	✓
Surgical wounds	✓	✓
Moist wound healing	✓	✗
Sterile	✓	✓
Can be cut to desired size	✓	✗
Suitable for fixing with Adhesive Foam Bandages	✓	✓

**snogg®**

**ANIMAL CARE**

**[www.snogg.com](http://www.snogg.com)**

Orkla Wound Care AB (Svetsarvägen 15),  
Box 1336, SE-171 26 Solna, Sweden  
Tel +46 (0)8 590 96325,  
[snogganimal@orkla.com](mailto:snogganimal@orkla.com)



620001370 V.01 2026, EN